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The screenshot displays the ADT interface with a patient admission notification overlay. The notification shows the room number '1bed1', a red warning '???Not Admi...', and a 'Check Settings' button. It also displays vital signs: 'HR 60' and 'SpO2 95'. Below the notification, the 'Patient Demographics' form is visible, including fields for Last Name, First Name, MRN, Visit Number, Gender, Date of Birth, Age, Height, and Weight. A 'HIS Action' dialog box is open, asking 'Confirm patient New2, Patient2 to 1bed1?' with 'Confirm' and 'Delay' buttons. The bottom navigation bar includes 'Patient Window', 'Manage Patient', 'Measurements', 'Review', 'Main Setup', and 'Main Screen'.

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Manage Patient

The screenshot displays the 'Manage Patient' window with a patient admission dialog box open. The dialog box, titled 'HIS Action', contains the text 'Readmit Mouse, Minnie?' and two buttons: a blue 'Readmit' button with a bed icon and a 'Delay' button. The background interface includes the following sections:

- Patient Demographics:** Fields for Last Name, First Name, MRN, Visit Number, Gender, Date of Birth, Age, Height (cm), and Weight (kg). A 'Find Patient...' button is at the top.
- Patient Care Information:** Fields for Equipment (7BMX550), Profile (Bedside), and Category (Adult). A 'Group' dropdown and 'Screen Notes' field are also present.
- Temporary:** Buttons for 'Transport/Standby...' and 'Resume'.
- Permanent:** Buttons for 'Transfer...', 'Discharge...', and 'Clear Sector...'.
- Buttons:** 'Apply' and 'Cancel' buttons are located at the bottom right of the main window.
- Navigation Bar:** At the bottom, a bar contains buttons for 'Patient Window', 'Manage Patient' (highlighted with an orange border), 'Measurements', 'Review', 'Main Setup', and 'Main Screen' (highlighted in blue).